

⌘ **HEINE DELTA® 20T Dermatoscope**

Precision optics combined with the ability to toggle between polarised and non-polarised illumination



Highest Quality Images and Diagnostic Flexibility

The **DELTA 20T Dermatoscope** offers unrivalled precision optics and the functionality to toggle between polarised and non-polarised illumination. No time consuming exchange of contact plates is necessary, at the same time contact dermatoscopy with immersion liquids is possible at any point, allowing the examination method to be matched with the respective pigmented skin lesion to obtain the best diagnosis.



'Quality' has become one of most overused and misused terms today in an effort to sell products. Few companies, however, are able to tangibly define their products quality in terms of end-user benefits and value. At HEINE we clearly define the benefits of HEINE Quality in terms of Performance, Function and Durability:



PERFORMANCE

Precision HEINE Optics. High quality, achromatic optical system provides unmatched sharpness and resolution. Focusing system provides for 10x to 16x magnification for recognition of even the smallest details and structures.

There's LED. And there's LED^{HQ}. HEINE has set a new standard which maintains that only the best is good enough, from the selection of materials to processing, from light intensity and thermal management to a colour rendering index as high as possible. **That's what we call LED in HEINE Quality – or LED^{HQ}.**



LED IN HEINE QUALITY.



FUNCTION

Toggle between polarised and non-polarised illumination modes. Conveniently alternate between polarised and non-polarised views with the press of a button – no exchange of contact plates necessary. Allows recognition of the "Blink Sign" when observing crystalline structures and presence of milia cysts, increasing diagnostic function.

Optional small contact plate available. Special small contact plate for difficult to access lesions.

Digital documentation capabilities. Connect to major brand SLR digital cameras with HEINE digital camera lens system. Specially designed lenses integrate the camera and DELTA 20T optics together to provide the highest quality imaging possible.




DURABILITY


5 Year Guarantee. HEINE makes no compromises in manufacturing high quality medical instruments. Our commitment to vertical integration in manufacturing means we control all aspects of our instrument quality.

LED Thermal Management. LED illumination systems require thermal management to ensure long-term performance and product stability. Carefully selected and applied heat-diffusing materials in the DELTA 20T guarantee consistent quality and performance throughout the LED's lifetime rating of up to 50.000 hours.

"Blink Sign" observable in a basal cell carcinoma

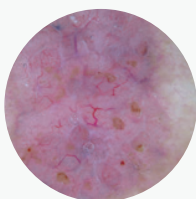

Basal cell carcinoma
non-polarised illumination

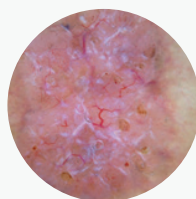

Basal cell carcinoma
polarised illumination

